# Dispatch® Hospital Cleaner Disinfectant with Bleach

Optimizing the use of Dispatch® products in hospitals

For more information, contact your Clorox representative or call (800) 234-7700. email: [healthcare@clorox.com](mailto:healthcare@clorox.com) visit us: [www.cloroxprofessional.com](http://www.cloroxprofessional.com)

1. REMOVE gross soil if visible. For *C. difficile* spores, always clean surface prior to disinfecting.

2. POUR directly onto hard, nonporous surface or a clean, dry cloth or Microfiber cloth. Wipe surface with cloth until completely wet. Gloves should be worn.

3. To DISINFECT, let stand for 1 MINUTE. To kill TB, allow 30 seconds contact at room temperature. To kill *C. difficile* spores, allow 5 minutes contact. WIPE with a clean, damp cloth. Allow to air dry.

**Commercial  
SOLUTIONS®**

© 2011 Clorox Professional Products Company  
1221 Broadway, Oakland, CA 94612

NI-15708

## Applications: Medical

Hard, nonporous hospital surfaces such as autoclaves, blood glucose monitors, external surfaces of medical equipment, IV poles, exam tables, bed railings, external surfaces of dialysis machines, gurneys, mammography equipment, plastic mattress covers, phlebotomy trays, reception counters, external surfaces of respiratory equipment, stretchers, external surfaces of ultrasound transducers and probes, wash basins and wheelchairs.

## Use Sites: Medical

Ambulance, central service, critical care, dialysis clinics, emergency rooms, exam rooms, home healthcare, hospitals, intensive care, laboratories, neonatal nursery, patient rooms, physical therapy rooms, physician offices, radiology, recovery rooms, respiratory therapy and surgery.

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